


Kronner Foot Casting Stand

Instructions for adjusting the foot plate angle

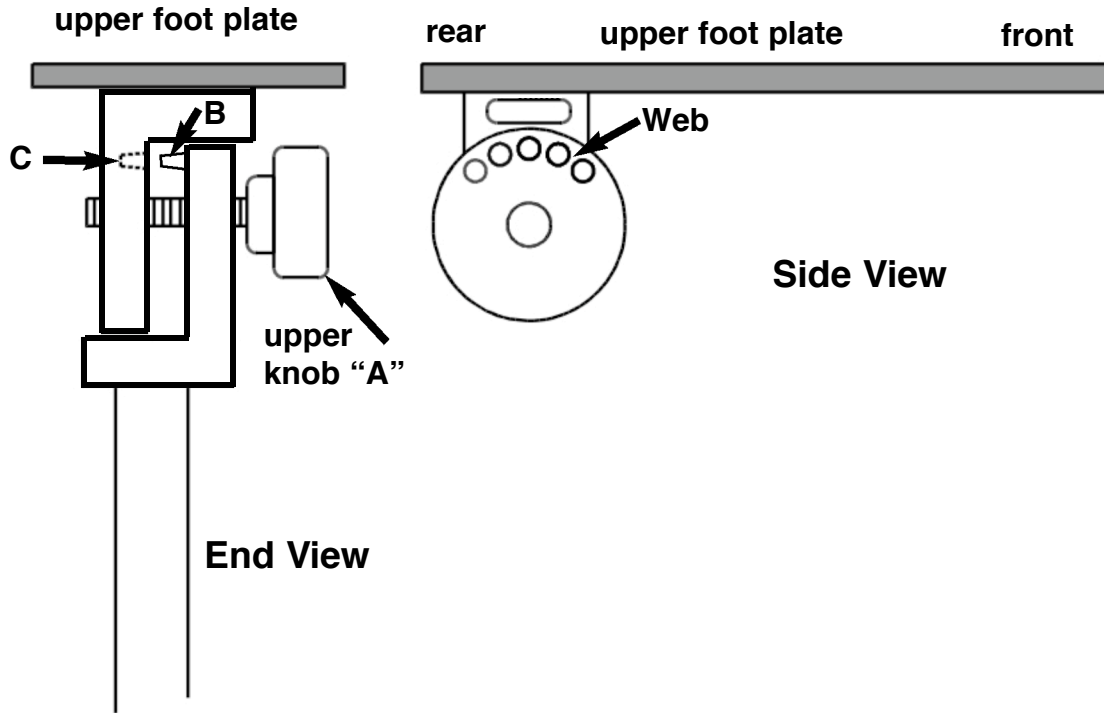

To adjust the angle of the upper plate of the footstand, release the upper knob "A" until the tapered pin "B" is completely out of recess "C".

Pull the two components apart and position the new plate angle. Do not tighten the upper knob "A" with the insert "B" positioned over the webs between the recesses.

All accessories attach to the **rear** of the upper foot plate except the **leg support**, part number 2-360, which attaches to the front of the plate.

Kronner Medical

1443 Upper Cleveland Rapids Road

Roseburg, OR 97471

Tel: 541-672-2543

FAX: 541-672-1074

Web: www.kronner.com