


# The Kronner Sidekick

## A Uterine Manipulator and Instrument Holder

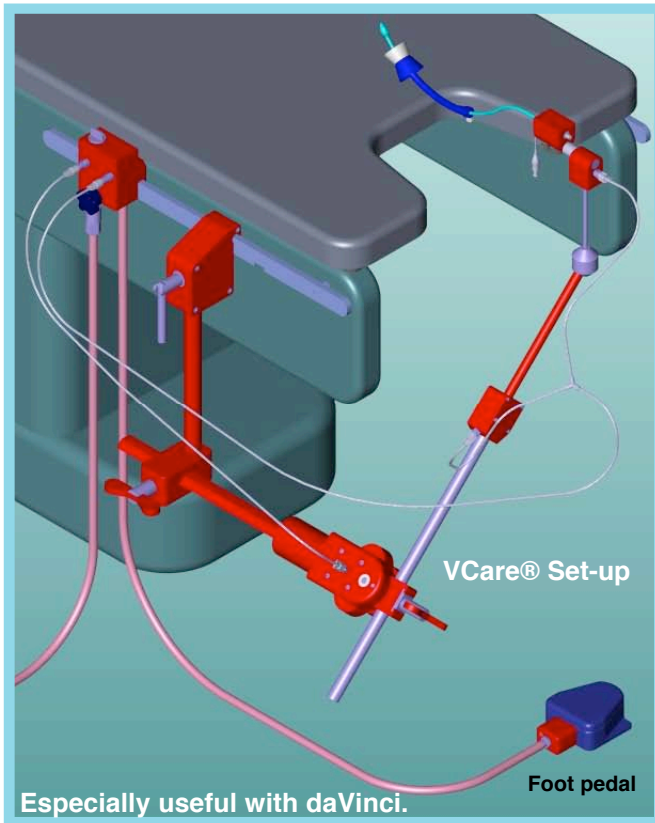


The **Kronner SIDE-KICK**, firmly holds uterine manipulators during laparoscopic surgery. Pressing a foot pedal releases all Sidekick joints for re-positioning the uterus. Holding the manipulators with this device reduces personnel fatigue.

Side-Kick is attached to either side of the operating table rail behind the leg stirrup attachment. The manipulator is inserted into the patient using standard technique, then Side-Kick is joined to it.

With proper adapters and telescoping rod Side-Kick may be used with manipulators such as RUMI®, the Kronner Manipjector®, other types, and also a perineal retractor. VCare® uses its own rod and doesn't require a separate adapter.


A gas supply module, which can be attached to the operating table rail, transfers gas from the wall or a portable tank to Side-Kick through two disposable lines. The lines are supplied sterile.

Under most circumstances only the arm assembly and adapters are sterilized. They may be steam autoclaved.

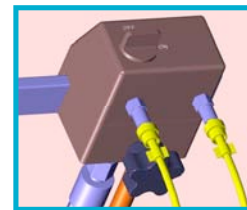


**Gas module**


### ADAPTERS

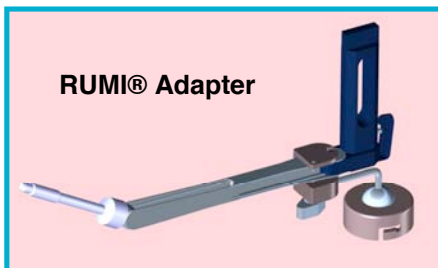


**RUMI® Adapter**

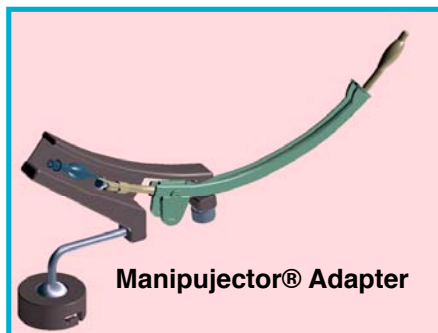


**Manipjector® Adapter**

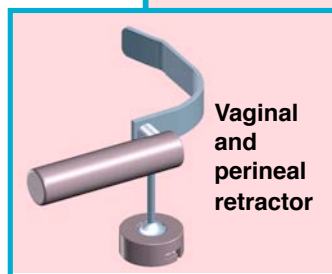

**Vaginal and perineal retractor**

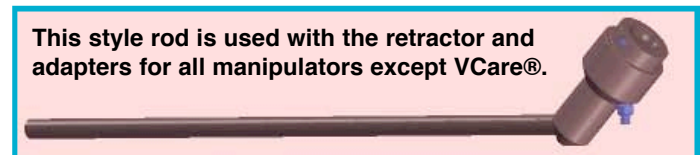

**This style rod is used with the retractor and adapters for all manipulators except VCare®.**

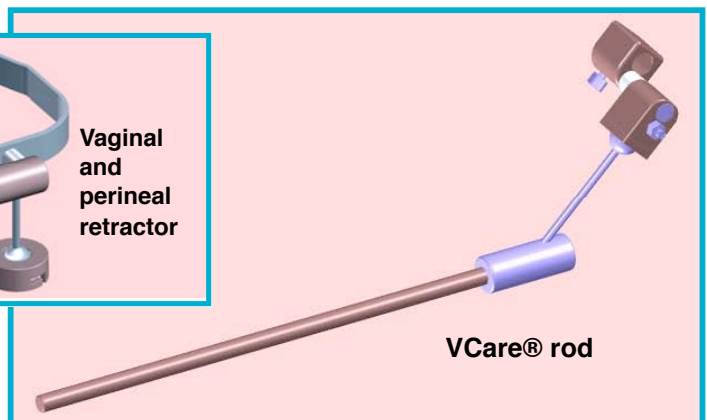

**VCare® rod**

**Kronner Medical, 1443 Upper Cleveland Rapids Road, Roseburg, Oregon 97471**

Phone: (800) 706-3533 or (541) 672-2543 Fax: (541) 672-1074

E-mail: [kronner@rosenet.net](mailto:kronner@rosenet.net) Website: [www.kronner.com](http://www.kronner.com)

Patent Pending